


# SEALBOND ULTIMA®

## UNIVERSAL LIGHT-CURE ADHESIVE BONDING AGENT

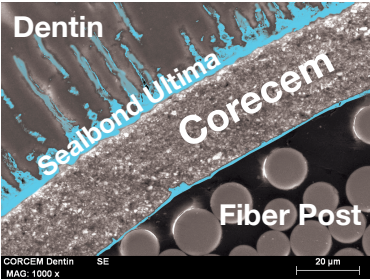

Post / Sealbond Ultima / Corecem Interface



SEALBOND ULTIMA is a 5th generation bonding system in order to present the best proven results for post cementation. SEALBOND ULTIMA is compatible for use with light-cure, dual-cure and chemical-cure cements and composites. SEALBOND ULTIMA is highly light-sensitive. It works perfectly with our translucent fiber posts and our CORECEM Dual Cure Cement and Core Build Up.



Sealbond Ultima 1030410


Features	Benefits
5th generation, one-bottle formula means NO portioning, NO mixing, NO waiting!	Universal light-cure formula is chemically compatible with all Bis-GMA and UDMA-based resin cements and composites (light-cure, dual-cure and self-cure)
Low film thickness (~14 microns) allows use in direct AND indirect (cementation) procedures.	Developed for the popular, well-documented TOTAL-ETCH, MOIST BONDING technique
Only TWO coats required; saving time and material, and light-cures in ~10 seconds.	Provides high bond strength (23 MPa etched dentin / 30 MPa etched enamel), prevents micro-leakage.
Light –sensitive; making it superior for post cementation	Two-year shelf life, allows liberal inventories, peace of mind

SHEAR BOND STRENGTH USING ULTIMA

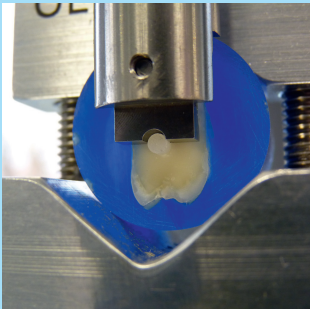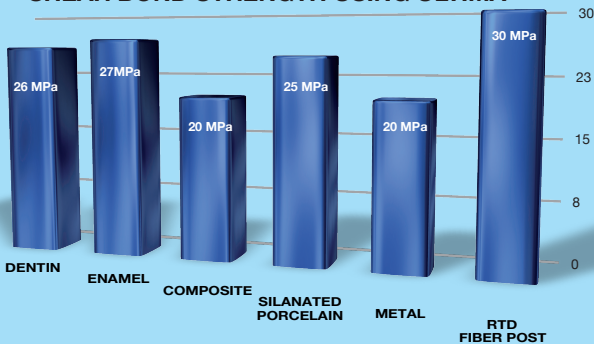


Dentin / Ultima / Composite Shear Bond Strength

RTD FIBER POST WITH DIFFERENT BONDING SYSTEM  
( INTERNAL DATA )

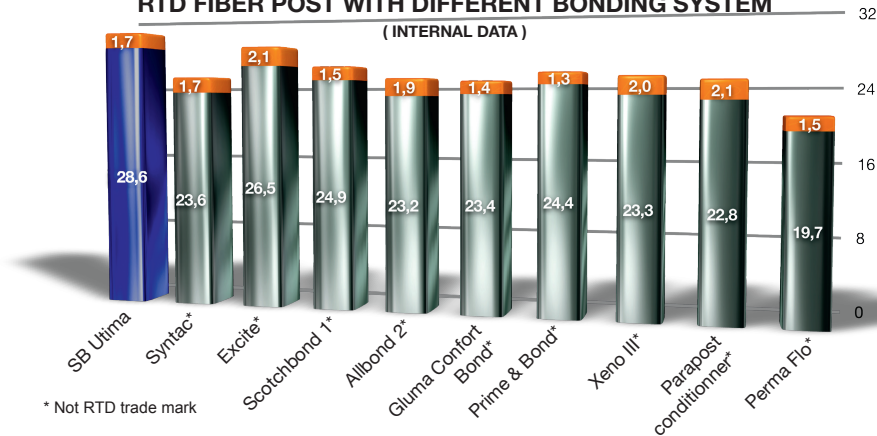


\* Not RTD trade mark

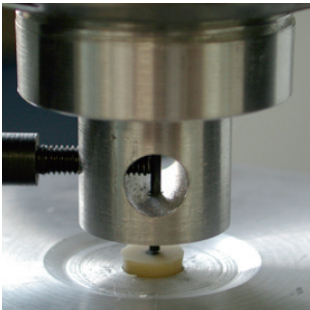

Composite / Primer / Fiber Post Push Out

## MACRO-LOCK ILLUSION X-RO CLINICAL CASE

*Courtesy Dr Alejandro Bertoldi Hepburn, University of Desarrollo, Chile*

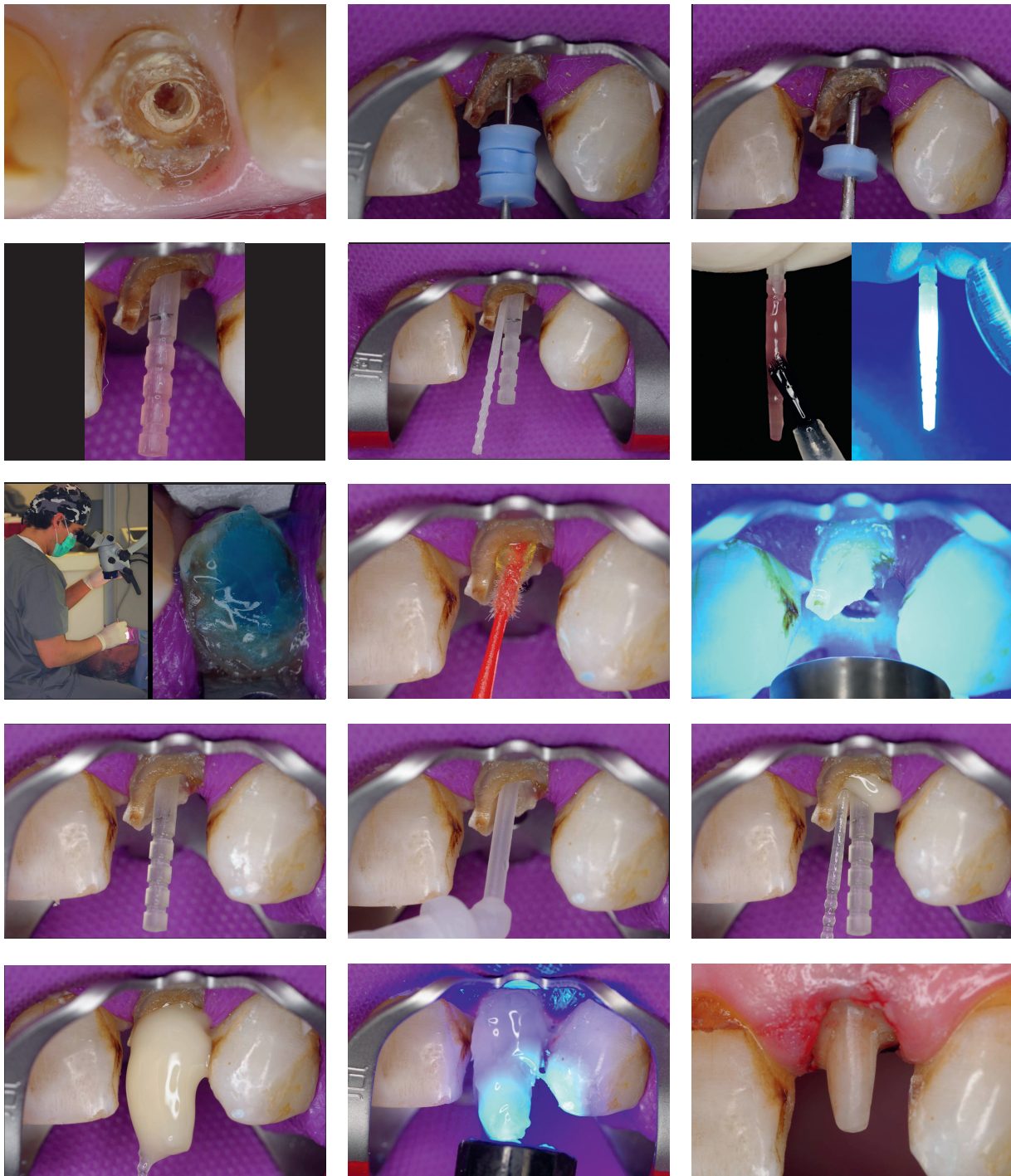

3 rue Louis Neel  
38 120 ST EGREVE - FRANCE  
Tél.: + 33 (0)4 76 56 56 66 Fax: + 33 (0)4 76 56 11 77  
E-mail: [rtd@rtd.fr](mailto:rtd@rtd.fr) / [www.rtd.fr](http://www.rtd.fr)

