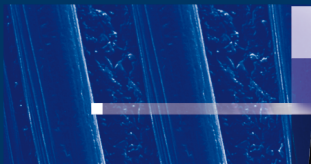
SEALBOND ULTIMA

INSTRUCTION FOR USE

MODE D'EMPLOI

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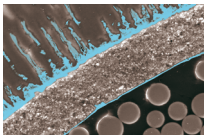
INSTRUCTIONS FOR USE

DESCRIPTION

Single component, 5th generation light-cured bonding system, with ideal film thickness for post cementation. Does not interfere with polymerization of self-cure or dual cure cements in self-cured mode.

INTERFACE DENTINE/CEMENT/POST

COMPOSITION

Methacrylate monomers with acetone solvent

INDICATIONS FOR USE

- Cementing fiber posts prior to crown reconstitution

CONTRAINDICATION

If the stipulated working technique cannot be used, the material should not be applied.

CAUTION

- Use an isolating technique such as rubber dam etc... as saliva contamination will compromise dentin bonding.
- Avoid contact between Sealbond Ultima and skin, mucous membrane and eyes. If contact occurs with eye, rinse immediately with water and seek medical advice if necessary. Wash with water and soap if contact occurs with skin.
- The use of protective gloves, safety glasses and mask are recommended.

SIDE EFFECTS

Unpolymerized resins may cause skin sensitization in susceptible persons.

OTHER INTERACTIONS

Do not clean the canal using hydrogen peroxide as it may inhibit bonding.

INSTRUCTIONS FOR USE OF SEALBOND ULTIMA

To the canal, apply a 32% phosphoric acid etchant of your choice for 15 seconds and rinse with water.

Blot the canal with a paper point to remove excess moisture. Leaving dentin and enamel visibly moist with a shiny surface.

POST SPACE PREPARATION:

Using a thin applicator, apply 2 consecutive coats of Sealbond Ultima to the canal and coronal dentin without waiting between coats.

Evaporate solvent and water with an air syringe (air dry) for 10 seconds.

Remove excess of primer with paper point.

Please note: the paper point step is important for removal of any pooled adhesive so that there is no interference in complete seating of the post.

Position of the light as close as possible to the canal opening. Light access is necessary for curing Sealbond Ultima in the canal.

Light cure the root canal and coronal dentine for a minimum of 10 seconds (500mW/cm²).

POST CEMENTATION:

Coat the post with Sealbond Ultima and lightly, air dry, and light cure for 10 seconds at 500mW/cm².

Express the dual-cured or self-cured cement of your choice directly into the canal working in an apical to coronal direction.

Quickly seat the post into the canal. Maintain firm pressure for 5-10 seconds once the post is seated.

Remove excess cement and light-cure for 40s to 60s at 500mW/cm² by placing the light tip on the post.

Once the cement has set, proceed with core build-up.

STORAGE

- Store product at room temperature (20-25 °C / 68-77°F)
- Do not use products after the expiry date displayed on the packaging.
- Keep away from sunlight

RECOMMENDATIONS

Immediately recap bottles after using to avoid solvent evaporation.

SAFETY AND LIABILITY

WARRANTY

RTD warrants this product will be free from defects in material and manufacture. RTD makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and RTD's sole obligation shall be repair or replacement of the RTD product.

LIMITATION OF LIABILITY

Except where prohibited by law, RTD will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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